

Triglyceride (TRIG)

Source:

Cerba Lancet Africa

View our website:

cerbalancetafrica.com

Date & Time of Export:

27 June 2026 10:34



[Click here to view the results online](#)

TEST OVERVIEW

Test Name	Triglyceride
Test Code	TRIG
Short Description	TRIG

OVERVIEW

Test Name	Triglyceride
Test Code	TRIG
Category	Biochemistry
TAT	Main Lab: 6 Hour(s) Family Site: <8hrs, <6hrs
Specimen(s)	1 x Venous blood - 5 mL Tube - Gold - SST-Serum Separator Tube

SPECIMEN(S)

SST-Serum Separator Tube

Specimen Type	SST-Serum Separator Tube
Specimen Format	Tube
Specimen Colour	Gold
Specimen Volume	5 mL
Sampling Order	2
Origin	Venous blood
Collection time after baseline	-
Transport Temperature	15-25°C
Accepted Other Specimens	Serum Lithium Heparin Plasma Sodium Heparin Plasma

TAT

EDTA Plasma

Main Lab: 6 Hour(s)
Family Site: <8hrs, <6hrs

Test Stability

Room Temp: 2 Day(s)
2–8°C: 7 Day(s)

PATIENT INFORMATION

Clinical Information Required

Fasting patient
Monitoring as part of treatment

Patient Collection Notes

In the absence of known hypertriglyceridaemia, a lipid profile may be taken in non-fasting patients. If hypertriglyceridaemia is discovered, a control sample should be prescribed and taken after a 12-hour fast. In all cases, the patient may drink water.

COMMENTS & NOTES

LOINC Code

927-8, 14927-8

Outwork

No